

## SAFE, EFFECTIVE, EASY TO USE

### How Do I Apply NewGel+?

First, ensure scar area is clean, dry, and free from creams, lotions, ointments and powders.

### NewGel+ Sheeting, Strips & Shapes:

- Peel off clear plastic liner and apply directly on scar.
- Wear daily for a minimum of 6-24 hours per day.
- Use daily for a minimum of 2-3 months.
- Washable and reusable for up to 4 weeks. Simply hand wash with water and mild soap, rinse well, and air dry.



### NewGel+E Gel:

- Apply a thin layer of gel and gently massage.
- Allow to dry for 1-2 minutes.
- Apply twice daily for a minimum of 2-3 months.



**For Optimal Results:** Use NewGel+ Sheeting for 12+ hours daily and apply NewGel+E when the sheeting is not being used to provide 24 hour-a-day coverage. Do not use the gel under the sheeting or the sheeting will lose the ability to stick to your scar.

Do not use NewGel+ on an open wound. Scar site must be completely closed (sutures removed, no scabbing or oozing).

See product insert for more information

## NEWGEL+ PRODUCT OPTIONS

BEIGE / CLEAR

12.7cm x 15.2cm Sheet (1 per box) NG-101 NG-301	2.5cm x 15.2cm Strips (4 per box) NG-101S NG-301S
Areola Circles (1 pair per box) NG-120	5.1cm x 20cm C-section (2 per box) NG-160
Anchor (1 pair per box) NG-126	Breast Kit (1 pair per box) NG-128
Lollipop (1 pair per box) NG-124	5.1cm x 61cm Strip (1 per box) NG-164
NewGel+UV SPF 30 0.5oz/15g NG-8015	NewGel+E Advanced Silicone Gel 0.5oz/15g NGO-800 1.0oz/30g NGO-810

Clear is more flexible  
Beige is more durable  
Gel can be used under make up or sunscreen

**NewGel+**  
SCAR MANAGEMENT SOLUTION

# IMPROVE YOUR SCARS



CLINICALLY PROVEN TO REDUCE  
AND FADE NEW AND OLD SCARS!


TGA-Cleared for Scar Management  
[www.newgel.com.au](http://www.newgel.com.au)

Exclusively distributed by

**ssa** surgical  
supplies  
australia

[www.surgicalsuppliesaustralia.com.au](http://www.surgicalsuppliesaustralia.com.au)  
1300 294 649  
[sales@surgsa.com.au](mailto:sales@surgsa.com.au)

MADE IN USA  
FDA Cleared

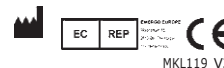

## CLINICALLY PROVEN TO OPTIMIZE AND FLATTEN NEW AND OLD SCARS

### What is NewGel+ Sheeting?

NewGel+ Sheeting is 100% medical grade silicone gel sheeting that helps soften, flatten, and fade scars. The variety of options, sizes and shapes to choose from are self-adhesive and worn directly on the scar.

### What is NewGel+E Gel?

NewGel+E Gel is a medical grade silicone, fast drying, non-sensitizing gel that softens, flattens, and fades scars. It is enhanced with 1% Vitamin E to increase the softening and moisturizing effect.

### How Does NewGel+ Work?

NewGel+ sheets and gel create a protective barrier over the scar, which reduces water evaporation and helps stop excessive scar tissue build up. This will help flatten and soften your scar, and reduce redness, itching and pain. NewGel+ is not absorbed into the body.

### How Soon Can I Begin Using NewGel+?

Use NewGel+ after sutures are removed, and the skin is fully closed. **DO NOT USE ON AN OPEN WOUND.**

### How Soon Will I See Results?

You will begin to see results in as little as 3 weeks. Best results are seen after 90 days.

## WHICH NEWGEL+ IS RIGHT FOR YOU?

### NewGel+UV SPF30 Silicone Gel

- Medical grade silicone plus zinc oxide
- Helps to protect new scars from hyperpigmentation cause by the sun
- Ideal for exposed areas such as face, neck, arms and hands
- No chemical sunscreens, only mineral



### NewGel+ Sheeting, Strips and Shapes

- Various shapes and sizes
- 2 backing options—clear and beige
- Flexible and durable
- Washable and reusable for up to 4 weeks
- Comes with a convenient storage tray

### NewGel+E Silicone Gel

- Soft, silky feel—not sticky
- Dries quickly
- Leaves no odor or residue
- Can be covered with sunscreen, cosmetics and clothing

### 3 STEPS TO IMPROVE SCARS

#### 1 Follow Your Doctor's Orders

Follow all take home instructions to ensure good wound healing and to prevent infection and inflammation, which may worsen scar appearance.

#### 2 Use NewGel+

Silicone gel and sheets are the #1 recommendation of scar experts as the preferred first-step treatment. NewGel+ offers the widest variety of medical grade products to meet the needs of every patient.

#### 3 Avoid the Sun

New scars should be protected from the sun because sun exposure may cause permanent hyperpigmentation, or darkening, of the scar.

AVAILABLE WITHOUT A PRESCRIPTION  
[www.newgel.com.au](http://www.newgel.com.au)

## PROVEN RESULTS: BEFORE & AFTER PHOTOS

### FACIAL TRAUMA



Before NewGel+



After 4 months of NewGel+ treatment

### PAEDIATRIC OPEN HEART SURGERY



Before NewGel+



After 4 months of NewGel+ treatment

### ACHILLES TENDON RUPTURE REPAIR



Before NewGel+



After 4 months of NewGel+ treatment

### MOHS SKIN CANCER SURGERY



Before NewGel+



After 7 weeks of NewGel+ treatment

## PDF Converter

Only two pages were converted.

Please **Sign Up** to convert the full document.

[www.freepdfconvert.com/membership](http://www.freepdfconvert.com/membership)